

Product Introduction

1.Product name

Car first aid kit

2. Brief introduction

The First aid kit can be used when the professional medical service cannot be available in time after the accident. The First aid kit includes some components, for example, absorbent non-woven compress, first aid dressing bandage, elastic bandage, adhesive bandage and etc. These components are assembled into a bag.

Classification of a first aid kit is determined by the medical device with the highest classification that is included. So, the First aid kit is Class Is.

3. Intended Use

It is used for temporary emergency treatment in case of accidental injury.

4. Models and specification of product

4.1 Model

DIN13164-2022

Item	Components	Raw materials/ Model	Specification	quantity	Sterilization Yes/No
1	First Aid Dressing Bandage	Bandage, nonwoven	6cmx3m/6cmx8cm	1	Yes
2	First Aid Dressing Bandage	Bandage, nonwoven	8cmx4m/8cmx10cm	2	Yes
3	First Aid Dressing Bandage	Bandage, nonwoven	10cmx4m/10cmx12cm	1	Yes
4	Absorbent non-woven compress	nonwoven	60cmx80cm	1	Yes
5	Absorbent non-woven compress	nonwoven	10cmx10cm-4p, 2 pieces	3	Yes
6	Elastic bandage	PBT	6cmx4m	2	No
7	Elastic bandage	PBT	8cmx4m	3	No
8	Triangular bandage	nonwoven	96cmx96cmx136cm	1	No
9	Adhesive Tapes	nonwoven	2.5cmx5m	1	No
10	Adhesive Bandage	nonwoven	51mmx45mm 2 pieces, 120mmx20mm 2 pieces 72mmx19mm 2 pieces 72mmx25mm 4 pieces 100mmx60mm 4 pieces	1	No



CE Technical File

First aid kit

File No: GK-CETF_1.1

Rev. No: A/1

11	First aid blanket	Gold blanket	1600mmx2100mm	1	No
12	Disposable vinyl Examination gloves	Vinyl	L,4 pieces	1	No
13	Scissors	Mental nickel plating	14.5cm	1	No
14	Cleaning wipe	nonwoven	12.5cmx18cm	2	No
15	Disposable medical mask	PP Spunbond nonwoven/ melt-blown nonwoven	17.5cmx9.5cm-3p	2	No

5.Contraindications

None

6.Sterilization

Absorbent non-woven compress and first aid dressing bandage are EO sterilization;

Other components are non-sterilization.

The whole package is non-sterilization.

7.Precautions

- 1) These components of the product are disposable product.
- 2) Do not use expired products.
- 3) Damaged inner packing is strictly prohibited.
- 4) Store in a dark, cool and dry place.

8.Warning

- 1)The product can be used only once, should be used immediately after opening the package and destroyed afterwards; if it is reused, the users or patients can be infected, patients have adverse reactions such as irritation and sensitization
- 2)Strictly prohibit using them if the single package is damaged. The product finked or affected with damp cannot be used. It should locate in the place where is abstained from high temperature and high humidity, and from caustic gases, and have good dryness and ventilation.
- 3)The period of validity is 5 years from the date of manufacture, must use them in this period
- 4)According to classification of medical waste, wastes of the production are Infectious wastes. They shall be classified and divided into packages or containers.
- 5) The product should be ensured that the dressing is properly tightened. If it is too loose, it cannot stop bleeding; If it is too tight, it will cause the blood circulation to be blocked; If the product is used as a bandage, make sure it is flattened and properly tightened.

9.Instruction for use

These components of the production have respective instructions, see First aid manual

10.Storage conditions

It is recommended to store in a dry, ventilated, non-corrosive gas environment at room temperature, away from ignition sources and flammable materials.

11.Transport Conditions

Sunscreen and rainproof during transportation

12.Validity period

5 years

13.Product pictures (Non-contractual pictures)



Fig.1 Components



Fig.2 Bag